The most common dermatological superficial skin indications in the practice which to be treated successfully by LIQUID FREEZING at intensive constantly penetration temperature of -89°C

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Indications	Indikationen	in sec	recom. freezetime	empf.Appl.Zeit	remarks	Bemerkungen
Aging pigmentation (in older persons the appearance of lentiform to coin sized brown spots especially on back of hands, flexor side of the underarms,face)	Altersflecke		2-3		(cosmetics indication) using a flat dermatological applicator according to size	
Lentigines (Naevi,lentiform,round or oval,brown spots, which as opposed to freckles are permanent	Lentigine		4-6		(cosmetics indication) flat or pointed applicator	
Acne nodulus (acne scars, appear during the course of acne)	Akneknoten		5-9		(cosmetics indication)	
Juvenile warts (flat epidermal papulae 3-4 mm in diameter, frequently in the face or on the back of the hands of juveniles)	juvenile Warzen		3-4		(cosmetics indication)	
			3-4		(cosmetics indication)	
Keratosis actinica (hornification caused by sun light)	Aktinische Keratosen		3-5			
Keratoma senile (Calluses due to old age, brown-black wart-like protuberances appearing especially in the face and back of the hand of older people)			7-8			
Basiloma	Basaliom		8-9			
Keloid (Bulging scar, hard, fibrous, sometimes itching star-shaped skin projections develop on scars after burns, acid burns, vaccinations or spontaneously	Keloide		5-7			
Naevus pigmentosus (Naevuszellnaevus with a nodulated, raspberry-like, furrowed surface)	Pigmentnävus		-7			
Acrochordon (skin tags)	weiche Fibrome		4-5			
Cavernous angioma	cavernöse Angiome		2-3		using by flat applicator due to compression	
Capillary hemangioma	kapilläres Hämangiom		1-2		therapying by flat applicator due to compression	

Verrucae vulgares (Common warts or spinous warts,maximally pea-sized, hemispherical hard nodules with a spiny surface,especially on the hands)	Gemeine Warzen	-11
Verrucae seborrhoicae (Warts appering in old age,usually appear after the 50 th birthday,light-brown to brown-black round to oval neoplasms)	Sebor.Warzen	-11
Plantar warts (Warts on the sole of the foot,spinous warts; their surface rarely protudes , their spiny projections grow into the depth, on the sole of the foot)	Plantarwarzen	-10
Molluscum contagiosum (Molluscum contagiosum, hard tumescence the size of a hemp seed to the size of a pea appearing especially in the face and genitals having a dent on top)	Mollusken	3-4
Erythematodes chronicus		3-5
Prurigo nodularis (Hemispherical to penny-sized hard, itching lumps with a rough surface, particularly on the extremitie s of older women)		-11
Lichen ruber (Small itching papulae which sometimes appear on the entire body on reddened skin)		3-6
Condyloma Acuminata (a sexually transmitted anogenital viral wart ,multiple, pointed, fissured growths, appearing in men,women and children)	Kondylom	3-12
Spinaliomas		6-8
Sarcomas	Sarkome	6-8
Eylidbasalioma	Augenlidbasaliom	3-4
Lid tumour	Lidtumor	3-4
Lip carzinoma	Lippenkazinom	3-4

There are only recommended application times. They are dependent of skin' s thickness and other factors. Tip: First try it on yourself! Place the device on the back of your hand for 1 sec and then for 2 sec.

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